1 Cemento-MP

Bestell-Nr. Order Code	Wiederverwendbares Applikationssystem inkl. Spüladapter und Paraffinöl	Reusable application system incl. flushing adapter and liquid paraffin
1382-0000		

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The powerful, reusable vertebroplasty system for quick, safe and effortless cement delivery, ideally suited for bone cement with high viscosity

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Cemento-MP, the reusable vertebroplasty system offers all the well known features of our established Cemento-RE system. The extra-large winged screw allows ease of use when applying high pressure, enabling exact and precise cement dosage in even the smallest quantities. The injection cylinder also features special reinforcements.

The high working pressure during cement delivery enables a fast injection and – in particular with inhomogeneous cavities and extremely porous vertebral bodies – a maximum fill . Even highly viscous cement can be used right up to its setting point. As a result, Cemento-MP creates a larger "window of opportunity" for the user to perform the procedure.

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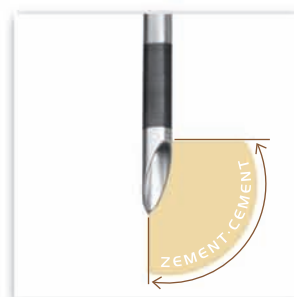
Special vertebroplasty needle featuring shock resistant, bayonet luer-lock fittings. The stylet and cannula feature a "tongue and groove" connection facilitating correct alignment, acting also as a needle-tip indicator. Special bevel-edge for precise steering of the needle and cement delivery.

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*Extra-large winged-screw applicator
for a powerful and controlled injection*

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